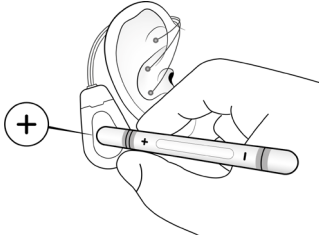
## ANWENDUNG VON VIVO & VIVO PEN

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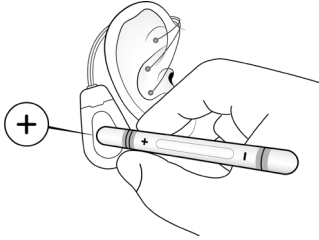
## VIVO & VIVO PEN OPERATION


### Increase stimulation intensity

Tap the stimulator with the "+" side of your VIVO Pen for 1 second.

- ✱✱ 2x green flashing light: stimulation intensity increased
- ✱✱ 2x orange-yellow flashing light: highest intensity reached


### Decrease stimulation intensity

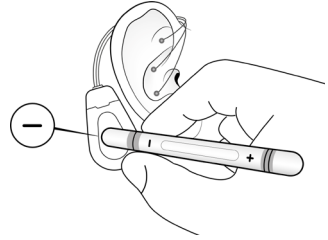Tap the stimulator with the "-" side of your VIVO Pen for 1 second.

- ✱ 1x green flashing light: stimulation intensity decreased
- ✱ 1x orange-yellow flashing light: lowest intensity reached

### Activate the stimulation during pause (min. 10 minutes)

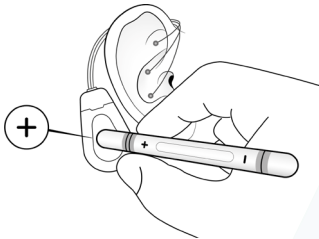Tap the stimulator with the "+" side of your VIVO Pen for 3 seconds.

- ✱✱✱ 3x orange-yellow flashing light: stimulation activated

### Malfunction

- Constant orange-yellow light or orange-yellow flashing light every 10 seconds:  
Please inform your physician.
- ✱ ... ✱

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## PLEASE NOTE THE FOLLOWING INFORMATION

VIVO stimulates the vagus nerve in your auricle in a fixed rhythm. VIVO Pen allows you to adjust the stimulation intensity in 36 levels. Please only use VIVO Pen for this purpose.

The stimulation should be perceived clearly and pleasantly, but not painfully.

Please comply with the operating conditions:

- Temperature: -10°C to 40°C
- Relative humidity: max. 90%

VIVO Pen can be wipe-disinfected with common disinfectants, e.g., 70% alcohol, and is intended for multiple use on a single patient.

In case of side effects, consult your physician. Serious incidents must be reported to the manufacturer and to the competent authority.

### DO NOT USE VIVO & VIVO PEN

- while swimming and in the sauna/solarium
- during clinical examinations (e.g., CT, MRI)
- in potentially explosive areas
- in areas of increased electric/magnetic field strength
- near sensitive devices/objects

### CONSULT YOUR PHYSICIAN IN CASE OF

- side effects (skin irritation of the neck and ear; pain, bleeding, numbness in the area of the needle electrodes; syncope; tachycardia; bradycardia; palpitations; fatigue/exhaustion; dizziness; nausea/vomiting; light-headedness; headache; high blood pressure; difficulty swallowing)
- malfunction of VIVO and VIVO Pen
- loosening/damage of needles/device

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